

Chapter 1 **EUKARE** Blood glucose monitor system

1.1 Introduction

Thank you for purchasing EUKARE blood glucose monitor system, this system can be used in both home and medical places, it is a delicate and precise blood glucose meter that helps diabetes patients detecting the glucose concentration in their blood.

Advanced bio electrode sensor technology is used in this system, only small amount of blood is needed to precisely calculate your blood glucose value and have it displayed on the display screen within 15 seconds. No need to count the time or wipe, it's just that simple to use.

Please read this manual carefully before you use the blood glucose monitor system, detailed drawings and descriptions are described 80 In the manual, you can easily get the information you need.

Applications: EUKARE blood glucose monitor system must be used together with **Ultra-speed** blood glucose test specimen when blood glucose concentration is examined in full blood. This equipment is suitable for diabetes patients to detect the blood glucose concentration in their own home by themselves; it can also be used in hospital or clinic as a tool to detect the glucose concentrations of the patients.



Warning: Please read this manual carefully and perform some practice before you use this blood glucose monitor system to measure blood glucose concentration. This system can only be used for blood glucose test reference; it should not be used as a medical diagnosis tool.

1.2 Package content

When you open this product's package, please check if all the followings items are in there, if any one of them is missing, please contact the distributor or change the product at the store you purchased it.

<input type="checkbox"/>	Items	Product name [Product serial number]	Quantity
<input type="checkbox"/>	1	EUKARE blood glucose meter [PM000301]	1
<input type="checkbox"/>	2	10 EUSURE blood glucose specimens, can-packaged [PS001001]	1
<input type="checkbox"/>	3	Setup and inspection card for EUSURE blood glucose meter [PA001201]	1
<input type="checkbox"/>	4	Blood picking tool XD [PA000403]	1
<input type="checkbox"/>	5	Blood picking needle 28 G XD [PA000301]	10
<input type="checkbox"/>	6	Li battery 3V[PA000101]	1
<input type="checkbox"/>	7	Medium level QC liquid [PA000601]	1
<input type="checkbox"/>	8	Nylon travel pack [PA000703]	1
<input type="checkbox"/>	9	EUKARE operational manual Chinese version (traditional Chinese)[PA001801]	1
<input type="checkbox"/>	10	EUKARE fast operational guide Chinese version (traditional Chinese)[PA001901]	1
<input type="checkbox"/>	11	EUSURE specimen manual Chinese version (traditional Chinese) [PA001102]	1











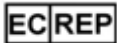

Optional (Not included in the package of this product, you can purchase them in the store you purchased this product if you need them)

- 50 EUSURE test specimen, box-packaged, Chinese version (traditional Chinese)[PS001206]
- High level QC liquid [PA000603]
- Low level QC liquid [PA000604]

- EUKARE transmission line [PA000202]
- EUKARE wireless receiver [PA002001]
- EUKARE blood glucose management software [PA002101]
- EUKARE blood glucose management software operational manual [PA002201]

一祥翻译公司 样本
Elegant Translation Service Sample
请勿复制
Do not copy

1.3 Product symbols and descriptions

	Don't repeatedly use		Please refer to the manual
	Keep dry		Warning
	Out-of-body Diagnosis		Temperature limit For specimen Preservation (4-30)
	Expiration date		Keep away from sunshine
	Lot number		Product serial no.
	European Community Representative		BF type equipment